GB PROCLIN® SOFT LENS MULTIPURPOSE SOLUTION with hyaluronic acid.
For cleaning, removal of protein deposits, disinfecting, rinsing, storing, inserting and moistening.
For soft contact lenses.

MEDICAL DEVICE

Composition:

Active Ingredient: Polyquaternium, Hyaluronate
Preservative: PHMB 0,0002%

Read the instructions and warnings carefully for proper use of the **PROCLIN® SOFT LENS** in combination with your contact lenses.

Recommended for sensitive eyes

Contains sodium hyaluronic acid that enhances the moistening of lenses and increases the tolerability of lens-wear due to its viscous-elastic properties. The solution hydrates lenses due to its 'water lock' not only after insertion but also throughout the day. It thereby aid the wearing comfort of the lens and diminishes eye irritation and dryness.

Precautions and instructions for use:

- Wash your hands carefully before removing the lenses from your eyes or the lens case. Dry your hands using a clean towel.
- Unsnapping the cap from the bottle will break the guarantee seal.
- Immediately after removal of the lenses, clean them with a few drops of **PROCLIN® SOFT LENS**. Clean the lenses by carefully rubbing them with your index finger in the palm of your other hand for 10-15 seconds.
- Rinse the lenses with **PROCLIN® SOFT LENS**.
- To disinfect the lenses, place them in the lens case filled with fresh **PROCLIN® SOFT LENS** and leave to stand for at least 6 hours. Remove the lenses from the lens case after the disinfecting cycle, rinse the lenses with **PROCLIN® SOFT LENS** and insert the lenses into your eyes.
- Rinse the lens case with **PROCLIN® SOFT LENS** and allow the lens case to dry in the air.
- Do not touch the dropper tip of the bottle as this may contaminate the solution.
- Close the bottle tightly immediately after use.

Other instructions:

After storage in excess of 7 days, lenses must be cleaned and disinfected again before wear.
Replace your lens case frequently to prevent infection due to contamination.

Warnings:

- Do not use if guarantee seal is damaged.
- Do not mix or use **PROCLIN® SOFT** with other contact lens fluids, unless it is indicated in the instructions for use.
- Always replace any remaining solution in the lens-case with fresh solution.
- Do not use after the expiry date.
- Do not change your contact lens system without consulting your doctor / contact lens specialist.
- Never use tap water as an alternative for this contact lens fluid.

Do not use:

If you are prescribed eye medication, consult your ophthalmologist or contact lens specialist before using the medication in combination with contact lenses.
In case of eye irritation, discontinue use and consult your doctor / contact lens specialist.

Store:

Keep out of the reach of children.
Store between 4°C and 25°C (40°F and 77°F).
Protect from sunlight.
Keep the solution in its original packaging.
Use within 3 months of opening.

Side Effects:

Do not use **PROCLIN® SOFT LENS** if you are allergic to one of the ingredients.

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TEGE Pharma B.V.